

Improved!

Kodak Min-R S Mammography Film

Quality mammography imaging for challenged processing environments

KODAK MIN-R S mammography film is our most robust film designed for optimal performance with shallow-tank and mixed processing environments. This green-sensitive, high-contrast film features stable processing tolerance even in non-optimized environments. The result is mammography image quality you can trust.



Fits your environment

KODAK MIN-R S film requires no adjustment to equipment or procedures in most cases. Compatible with all KODAK MIN-R screens, this film:

- ▶ Provides a 100-speed system with MIN-R screens and a 150-speed system with MIN-R 2000 screens.
- ▶ Provides good, consistent results with shallow-tank and shared processing equipment.
- ▶ Can be used with orthochromatic mammographic screens from other manufacturers.
- ▶ Provides optimal results when used with the complete KODAK system including processing solutions, screens, cassettes, and processors.

Fine detail perception

Images are sharp and provide viewing ease of diagnostic information. Perception of microcalcifications and other minute details is enhanced by the higher contrast and sharpness of the film.

New innovative design


Now with a pleasing blue tone, KODAK MIN-R S film is an innovative dual-emulsion film designed for use with a single intensifying screen. A proprietary technology employs two thin emulsions working in tandem to yield a superb image—while providing greater processing tolerance than a thicker single emulsion.

- ▶ The primary emulsion images breast parenchyma, providing detail and now even higher contrast.
- ▶ The secondary emulsion images the breast periphery and upper density.


A name you can trust

Carestream Health is a leader in mammography imaging because of our clinical expertise and commitment to your satisfaction. We combine state-of-the-art film technology with the right processing chemicals and equipment to provide the medical images you need for high-quality patient care.

Dual-emulsion technology

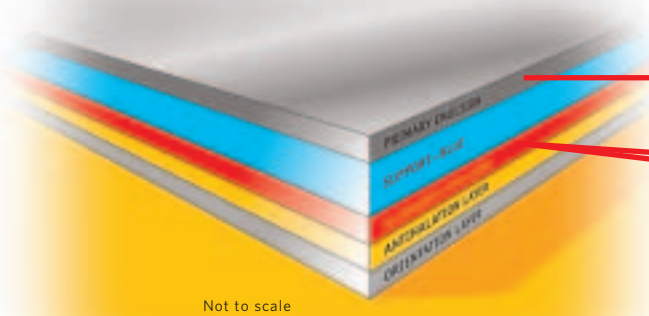

Not to scale

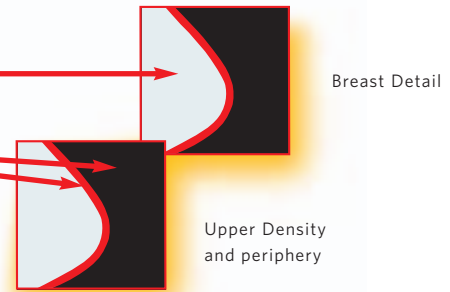

Ordering information

| PRODUCTS | CAT NUMBER |
|-------------------------------------|------------|
| KODAK MIN-R S film 24 x 30 cm | 173 2726 |
| KODAK MIN-R S film 18 x 24 cm | 151 3381 |

KODAK MIN-R 2 cassette with:

| | |
|--|----------|
| KODAK MIN-R 2000 screen 24 x 30 cm | 899 9492 |
| KODAK MIN-R 2000 screen 18 x 24 cm | 810 4101 |
| KODAK MIN-R 2190 screen 24 x 30 cm | 891 0853 |
| KODAK MIN-R 2190 screen 18 x 24 cm | 126 0579 |
| KODAK MIN-R screen 24 x 30 cm | 709 1614 |
| KODAK MIN-R screen 18 x 24 cm | 708 7984 |

KODAK MIN-R cassette with:

| | |
|--|----------|
| KODAK MIN-R 2000 screen 24 x 30 cm | 193 0247 |
| KODAK MIN-R 2000 screen 18 x 24 cm | 878 4449 |
| KODAK MIN-R 2190 screen 24 x 30 cm | 874 3171 |
| KODAK MIN-R 2190 screen 18 x 24 cm | 865 0426 |
| KODAK MIN-R screen 24 x 30 cm | 128 4413 |
| KODAK MIN-R screen 18 x 24 cm | 128 4397 |

Screen-film characteristics

| KODAK Screen | KODAK Film | Relative Processing Cycle | Relative Speed | Contrast | | D-Max |
|--------------|------------|---------------------------|----------------|----------|------|-------|
| | | | | RP | EXII | |
| MIN-R 2000 | MIN-R S | Standard | 150 | 3.80 | 4.0 | ≥ 4.0 |
| MIN-R 2190 | MIN-R S | Standard | 190 | 3.80 | 4.0 | ≥ 4.0 |
| MIN-R | MIN-R S | Standard | 100 | 3.80 | 4.0 | ≥ 4.0 |

Part of an unbeatable system

Use the MIN-R S film system with Kodak processing chemicals and equipment for mammography to help your facility maintain the highest standards of performance and care.

Kodak

Licensed Product

Carestream Health, Inc.
150 Verona Street
Rochester, NY 14608

More information

For details on KODAK MIN-R S mammography film, or to order, contact your Carestream Health representative or distributor, or call 1-877-865-6325, ext. 626.

→ www.carestreamhealth.com

Carestream 
HEALTH

The Innovation Powering **Kodak** Health Products